

## STIC TEMPORARY FILLING MATERIAL

### DIRECTIONS FOR USE

#### FEATURES:

A ready to use temporary filling material with a synthetic resin base. Adequate strength yet easily removed. Good adhesion to dentine and excellent marginal adaptation. Impermeable to medicaments, easy to use consistency.

#### INDICATIONS:

Temporary filling of cavities  
Temporary sealing of inlay preparations  
Temporary seal for medicaments  
Temporary fixing of post crowns

#### APPLICATION:

Using a suitable filling instrument, place direct into cavity and adapt as necessary. It is not essential to dry the cavity prior to placement. Temporary filling material hardens by absorption of moisture to give an impermeable seal. It is recommended that masticatory pressure is not applied for approximately 15 minutes. For provisional fixing of post crowns spread a layer on inside of crown and seat in normal way.

NOTE: For tube presentation, remove cap from tube, reverse and pierce membrane fully. Use key provided to assist extrusion from tube.

#### STORAGE:

Store in cool dry place (5 – 25°C). It is important that caps are tightly replaced after each use.

#### BATCH CODE:

The batch code gives an open date of manufacture in month, year, and day format, with a numerical suffix to uniquely identify the batch of material. This number must be quoted in all correspondence.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: 30g Jar (White)

Medicept Detal India Pvt. Ltd, 501, Bhakti Park, R.H.B. Road Mulund West, Mumbai 400080. Ph: +91 22 2562 9999

*Medicept Dental operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.*